



ALL SILICONE CLEAN INTERMITTENT CATHETERIZATION SET

All Silicone Clean Intermittent Catheterization Sets (C.I.C) are used for urine drainage for the patients who have problem that normal urination becomes impaired. This catheter should be used only for urology. C.I.C is a portable and reusable self- catheterization set.



● Features of Products

- Clean Intermittent Catheter is supplied in plastic case.
- Disinfectant prescribed by physician should be filled in plastic case.
- Available in male, female and pediatric type.
- Supplied with carrying case.

Directions for Use

Catheter Insertion

- Wash hands with soap before insertion of C.I.C.
- Open the blue cap and take the catheter out from the plastic case. Do not remove blue cap from the catheter funnel.
- Hold the catheter with fingers and insert it gently into the bladder through urethra.
- Remove the blue cap for urine drainage.
- After finishing urine drainage, remove the catheter and wash it with water. Then connect blue cap into catheter funnel and put the catheter back in the plastic case which is filled with disinfectant.

Warning

- Take out catheter from packaging and check whether or not the catheter has been damaged. If the catheter has been damaged, do not use it and report to supplier.
- Refill disinfectant everyday for better disinfection.
- Should follow doctor's instruction when using this product.
- Catheter should be sterilized by boiling water for 15 minutes every 2 – 3 days. Please do not boil blue cap and plastic case.
- Do not use disinfectant containing chlorohexidine or iodine. They can give damage to the catheter.

Suggested Disinfectant

- Mixed solution of 0.02% ~ 0.05% Benzethonium Chlorite with sterilized glycerin.

Prod. No	Type	Size (Fr)	I.D (mm)	O.D (mm)	Length (mm)
1035F-10	Female	10	1.7	3.3	165
1035F-12		12	2.0	4.0	
1035F-14		14	2.5	4.7	
1035M-10	Male	10	1.7	3.3	300
1035M-12		12	2.0	4.0	
1035M-14		14	2.5	4.7	
1035ML-12	Male, Long	12	2.0	4.0	350
1035PF-09	Pediatric, Female	9	1.5	3.0	150
1035PM-09	Pediatric, Male	9	1.5	3.0	265

- Do not expose the product to high temperature, humid air and ultraviolet ray during storage.
- Sterility guaranteed unless package has been opened or damaged. Do not re-sterilize.



Manufacturer:

Yushin Medical Co., Ltd.

36, Jomaru-ro, 427beon-gil, Wonmi-gu, Bucheon-si,
Gyeonggi-do, 420-837, Korea
TEL: 82 32 668 0323 FAX: 82 32 668 0329
www.yushinmed.com / ysmed01@yushinmed.com

Distributor: